




# Adjusting Appearance 1

**Proloquo2Go** (iPhone, iPad and iPod touch). In this short tutorial you will learn the basics for adjusting the appearance of Proloquo2Go.

## Changing Button Size

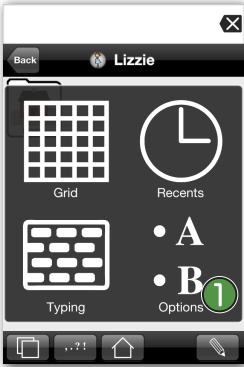

Launch Proloquo2Go by tapping on its icon. Go to the Options view.

On an iPod touch or iPhone, the Options View can be accessed by tapping the views toolbar button at the bottom left of the screen, and then tapping the Options button 1 in the Views display.



On iPad, the Options button 1 is the right-most button on the toolbar.




Tap on the Appearance section. 1




In the Display section, 1 tap on Item Size. 2 In this section, you can change the size of buttons and consequently the number of buttons on the screen.

Available item	# of Buttons	# k wide items
Extreme	1	2
Very Large	4	6
Large	9	8
Medium	16	12
Small	25	20
Very Small	36	30
Super Small	49	42
Ultra Small	64	56


In this overview you can see how many buttons display with which Item Size.

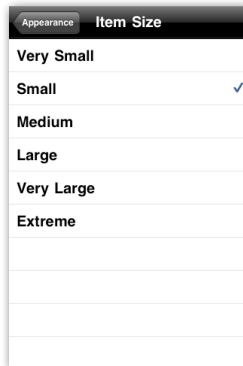

Select the Item Size you prefer.

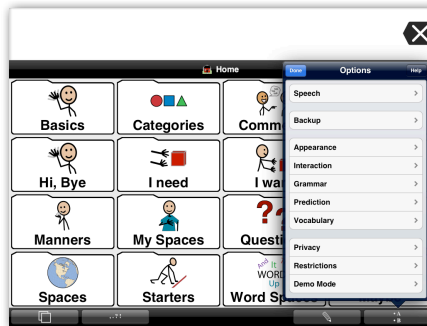

On iPad, you can see how the size of the items automatically changes.

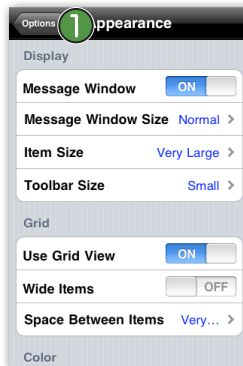

On iPod or iPhone, tap Options on top of the Appearance screen and then tap Done to exit the Options Mode. Now you can see the Item Size you have selected.



## Changing Toolbar Size



Go to the Options view. If you do not remember how to get there, please see the first two steps of this tutorial.

Tap on the Appearance section. **1**




In the Display section, tap on the Toolbar size item. **1** Select another size for the Toolbar Size.

*On iPad, you can see the changes immediately again. On iPhone or iPod touch, follow the instructions above to exit Options View.*

## Changing from Grid to List View



Go to the Options view and tap on the Appearance section. In the Grid section and toggle Use Grid View to OFF. **1**

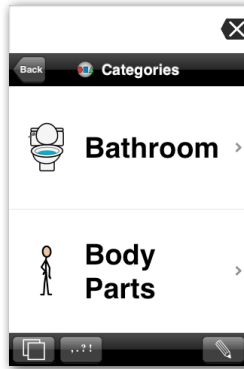

Note that Proloquo2Go is now in List View. Toggle the Use Grid View back to ON to switch back to Grid View.

## Hiding the Message Window



Go to the Options view and tap on the Appearance section. In the Display section, toggle the Message Window OFF. **1**




*On iPad, you can see the changes immediately again. On iPhone or iPod touch, follow the instructions above to exit Options View.*

Note how the Message Window is now hidden.

## Adjusting Wide Items



Go to the Options view and tap on the Appearance section. In the Grid section, toggle the Wide Items to ON. **1**

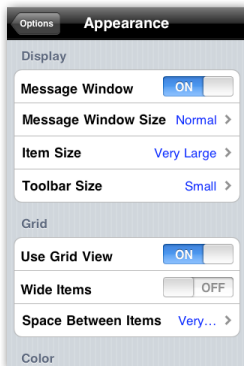

Note how the buttons have changed to a different width. Toggle the Wide Items back to OFF to switch back to the normal settings.