

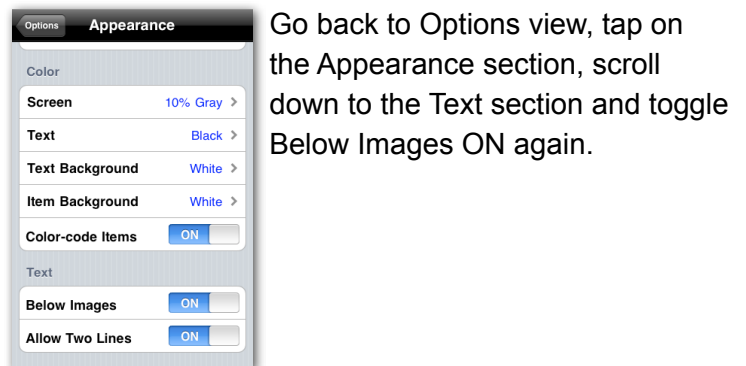
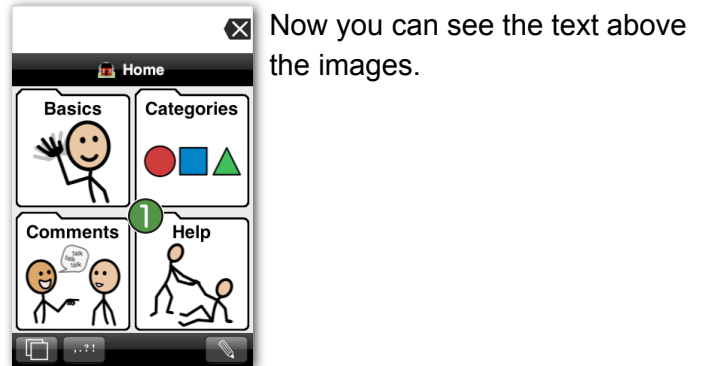
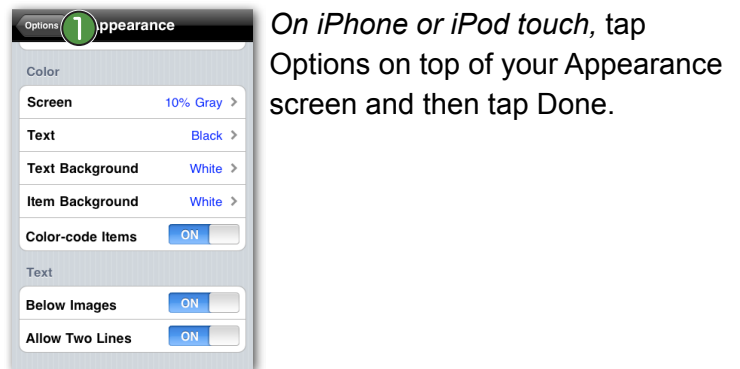
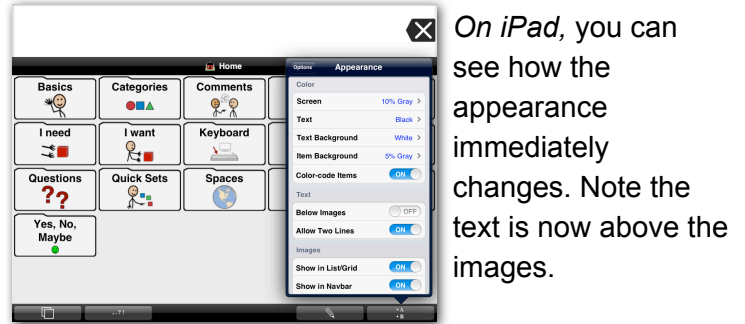
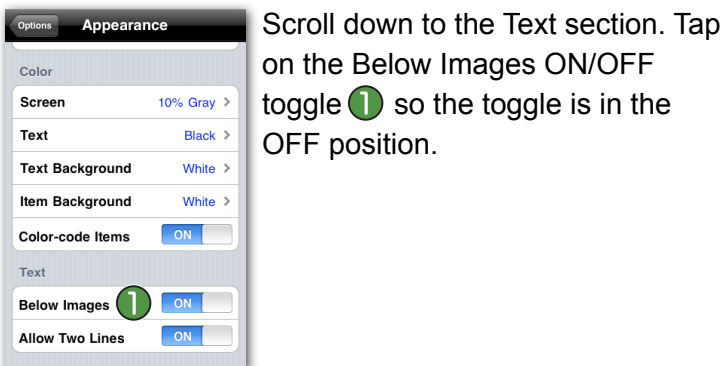
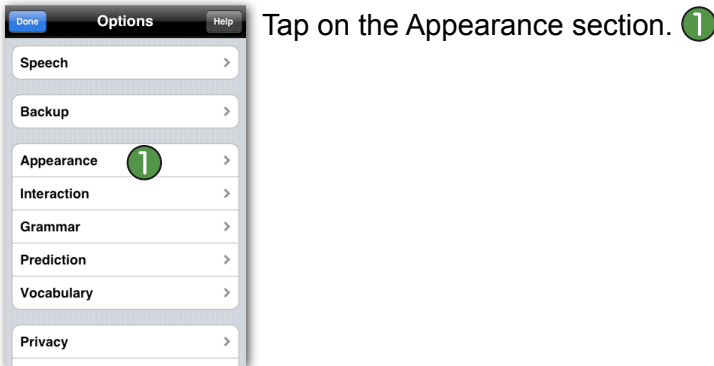


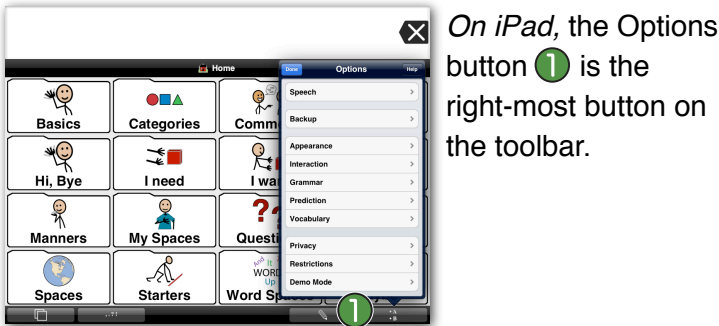

Adjusting Appearance 3 - Text

Proloquo2Go (iPad, iPhone and iPod touch): In this tutorial you will learn how to change the text location on an item and how to change text from 2 lines to 1 line.

Changing Text Location on an Item

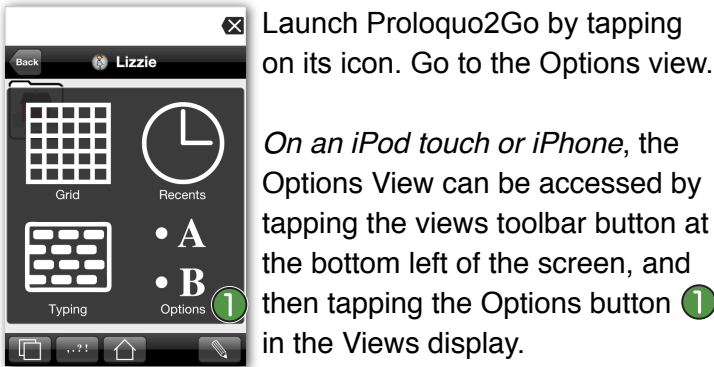




Changing Text from 2 lines to 1 line




If you are not already in Proloquo2Go, launch Proloquo2Go by tapping on its icon. Tap on the Home button on the toolbar. Notice that the label of the item Yes, No, Maybe **1** is shown in two lines of text.

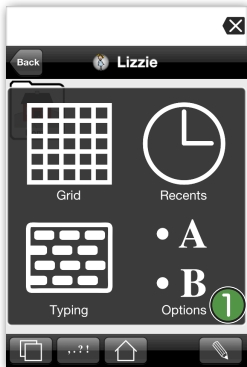

Go to the Options view.

*On an iPod touch or iPhone, the Options View can be accessed by tapping the views toolbar button at the bottom left of the screen, and then tapping the Options button **1** in the Views display.*

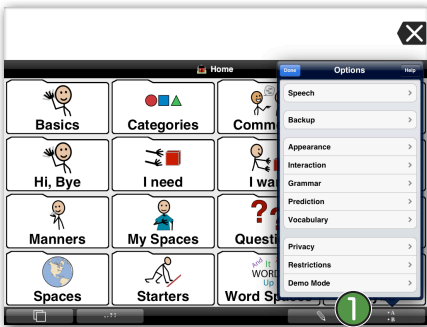

*On iPad, the Options button **1** is the right-most button on the toolbar.*




Tap on the Appearance section. **1**



Scroll down to the Text section. Tap on the Allow two lines ON/OFF toggle **1** so the toggle is in the OFF position.



On iPad, you can see how the appearance immediately changes. Notice that the label of the item Yes, No, Maybe is now fits on one line.

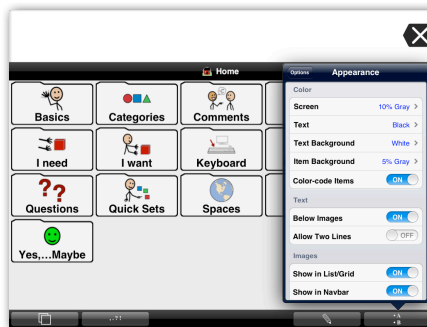

On iPhone or iPod touch, tap Options on top of your screen and then tap Done. Now you can see how Yes, No, Maybe is shortened to fit on one line.



If you want to change the settings back to normal, please follow the instructions above again.