




# Adjusting Restrictions

**Proloquo2Go** (iPhone, iPad, iPod touch) In this tutorial you will learn everything about restrictions to prevent the user from accessing certain features.



Launch Proloquo2Go by tapping on its icon. Go to the Options view.

*On an iPod touch or iPhone, the Options View can be accessed by tapping the views toolbar button at the bottom left of the screen, and then tapping the Options button 1 in the Views display.*

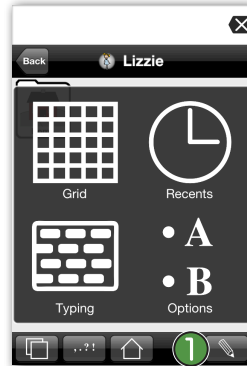


This will prevent the user to access the Edit feature 1 from the toolbar.




*On iPad, the Options button 1 is the right-most button on the toolbar.*




*On an iPad, you can see how the Edit button (the pencil) is removed immediately.*

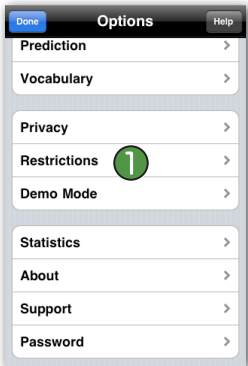

Tap on the Restrictions section. 1

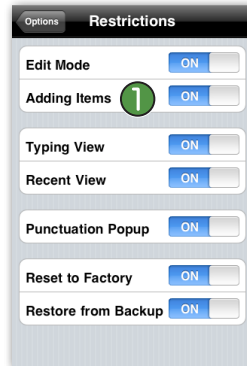


Tap on the Adding Items ON/OFF toggle so it is in the OFF position.

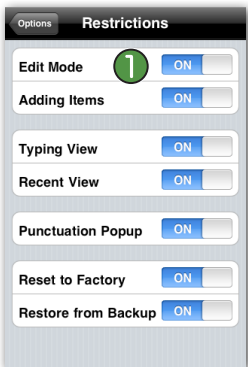


Tap on the Edit Mode ON/OFF 1 toggle so it is in the OFF position.



The plus button is now removed, so this will prevent the user to add new items.

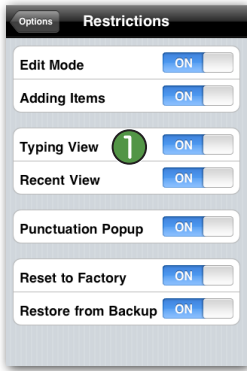


Tap on the Typing View ON/OFF toggle 1 so it is in the OFF position.

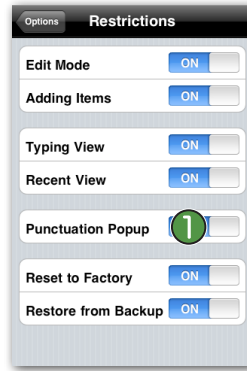


Tap on the Punctuation Popup ON/OFF toggle 1 so it is in the OFF position.



This will prevent the user from accessing Typing View when they tap on the Views button 1 on the bottom toolbar.

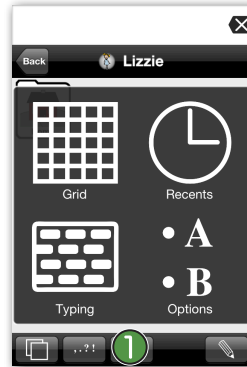


This will prevent the user from accessing the Punctuation Popup 1 from the bottom toolbar.

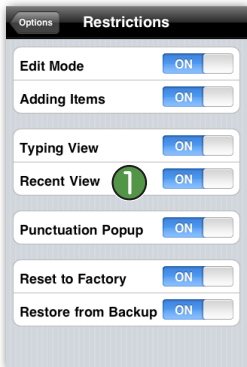


Tap on the Recent View ON/OFF toggle 1 so it is in the OFF position.



On an iPad, you can see how the Punctuation button 1 is removed immediately.

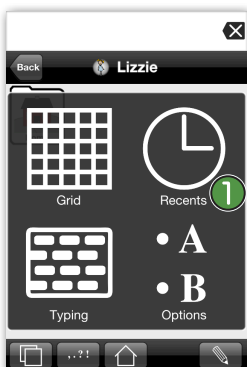

This will prevent the user from accessing Recent View when they tap on the Views button 1 on the bottom toolbar.

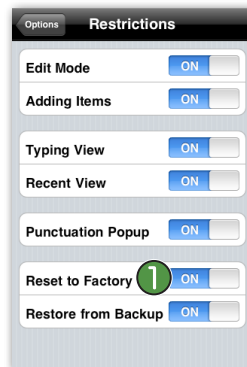


Tap on the Reset to Factory ON/OFF toggle 1 so it is in the ON position.




Now tap Options on the top of the Restrictions screen and go to Backup. Note how Reset to Factory Defaults 1 is now available. If Reset to Factory is OFF, the option is greyed out.




Now tap Options on the top of the Backup screen and go back to Restrictions. Tap on the Restore from Backup ON/OFF toggle 1 so it is in the ON position.

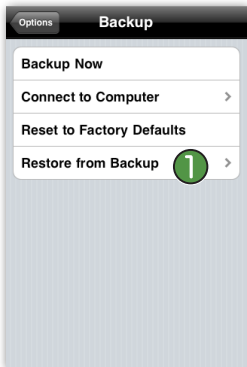

Now tap Options on the top of the Restrictions screen and go to Backup. Note how Restore from Backup 1 is now available. If Restore from Backup is OFF, the option is greyed out.

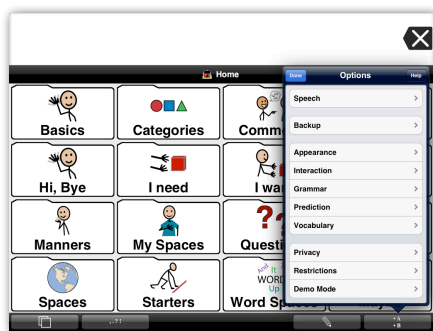


On an iPad, to exit the Restrictions, tap anywhere on your screen.

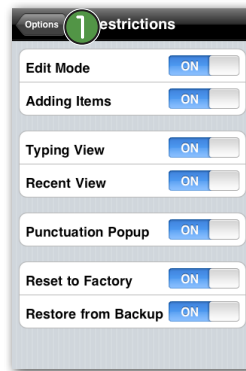


On an iPhone or an iPod touch, tap Options on the top of the screen and tap Done to exit Restrictions.



You will notice that the user does not have any access to the toolbar items. This prevents them from either intentionally or unintentionally adding or deleting items or accessing parts of the program in which they might get lost.

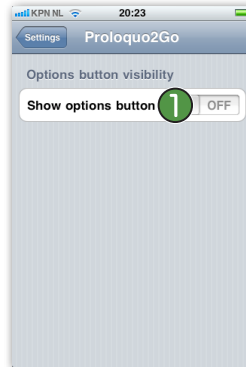

If you want to prevent the user to access all the options, you can restrict access to the Options View. To do so, exit Proloquo2Go by pressing your device's Home button, go to the Settings and scroll down to Proloquo2Go. Here toggle the Show options button 1 to OFF.

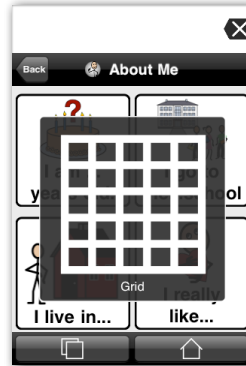

When you go back to Proloquo2Go and on an iPhone or iPod touch you tap the Views button, you will see that the Options button has been removed.

On iPad, the Options button is removed from the toolbar. The user is now locked in the Grid view.