





Modifying Pronunciation

Proloquo2Go (iPhone, iPad and iPod touch). In this tutorial you will learn all about modifying the pronunciation of a word. Moreover, we will go into using phonetics.

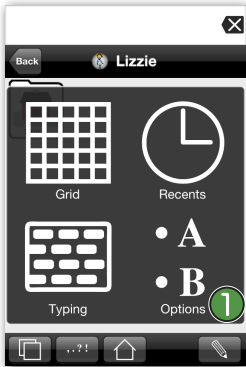

Launch Proloquo2Go by tapping on its icon. Go to the Options view.


On an iPod touch or iPhone, the Options View can be accessed by tapping the views toolbar button at the bottom left of the screen, and then tapping the Options button 1 in the Views display.

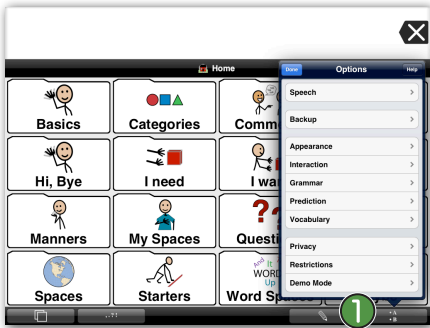

On iPad, the Options button 1 is the right-most button on the toolbar.



Tap on the Speech item. 1 Then tap on Pronunciation. This will take you to a screen showing a list of the words and terms for which a pronunciation correction is in place.

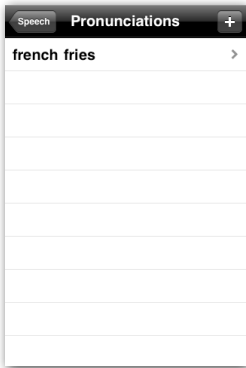


By default, for American English you will see the term "french fries". Tap this word and you will see the details for this term.



At the top you will see the original word. Below that the new pronunciation is displayed. To edit the contents of these fields briefly tap on the field so that a keyboard will appear allowing you to edit the text. Tap the Done button on the keyboard when done making modifications.




The Use Phonetics 1 is for advanced users and will be discussed in a bit. The Speak button 2 allows you to listen to the resulting pronunciation. The Delete button 3 allows you to remove the current pronunciation exception from Proloquo2Go.

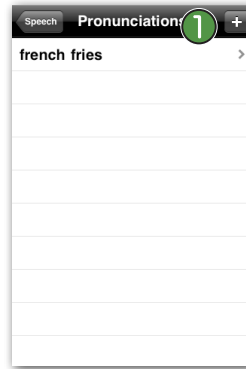


When done entering a pronunciation exception, press the "Pronunciations" button at the top of the screen to go back to the list of pronunciation exceptions.


Press the + button 1 at the top of the Pronunciations screen to add a new pronunciation exception.

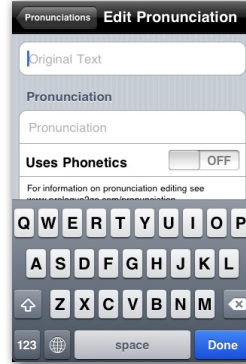

When you press the Add button, the Edit Pronunciation screen will appear. The keyboard will appear as well allowing you to immediately enter the original text and the pronunciation you want.

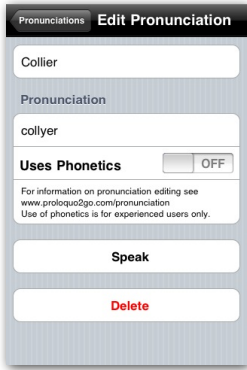

Type "Collier" in the Original text field and "collyer" in the Pronunciation field. Then press the Done button on the keyboard to hide the keyboard. This will reveal the Speak button so that you can test the pronunciation.

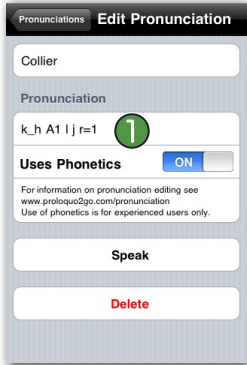

If you are not satisfied with the result you can try using phonetic codes for more control. This is for advanced users only as it is easy to make errors. Try replacing the text in the Pronunciation field with k_h A1 l j r=1. When you are done press the Done button on the keyboard.



Now switch the Use Phonetic button to ON. 1



If you made an error you will get an error message.



You need to fix the error before Proloquo2Go will allow you to switch the button to ON.



If you do not know where you made the error, enter the phonetic codes one by one and switch the Use Phonetics toggle to ON after each letter. You will receive the error message after entering the wrong phonetic code.



Once you are successful, you can press the Speak button to hear the result. 1



More information on the use of phonetic codes can be found in the Proloquo2Go manual, which can be downloaded here: <http://www.proloquo2go.com/files/Proloquo2GoManual.pdf>