




Adjusting Speech Options

Proloquo2Go (iPad, iPhone and iPod touch). In this tutorial you will learn all about the speech options: Voices, Speech Rate, Voice Personalization, Speak Message Only, Speak All Items and Pronunciation.

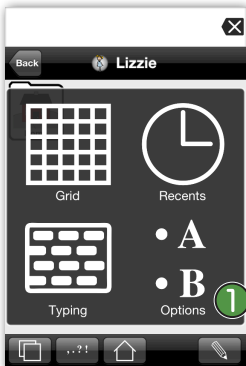

Launch Proloquo2Go by tapping on its icon. Go to the Options view.


On an iPod touch or iPhone, the Options View can be accessed by tapping the views toolbar button at the bottom left of the screen, and then tapping the Options button 1 in the Views display.



On iPad, the Options button 1 is the right-most button on the toolbar.




Tap on Speech. 1

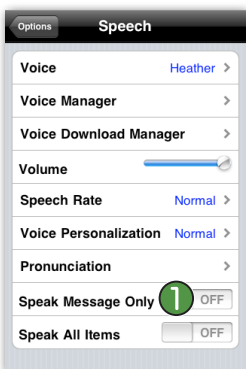


Tap on the Speak Message Only toggle 1 so it is in the ON position.


This makes Proloquo2Go only speak when the Message Window is tapped.

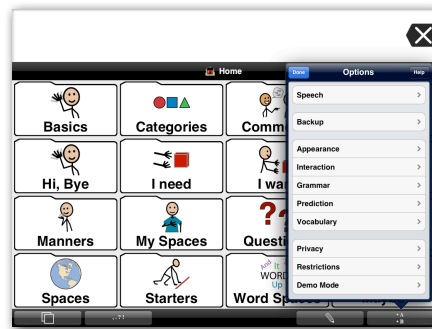


On an iPad, tap anywhere outside the Speech screen to exit Options.

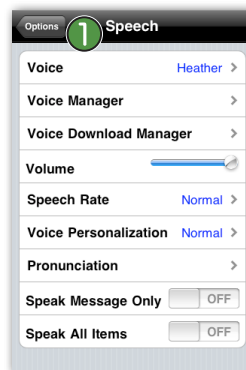

On an iPhone or iPod touch, tap on Options on the top of the Speech screen 1 and then tap Done to exit Options.

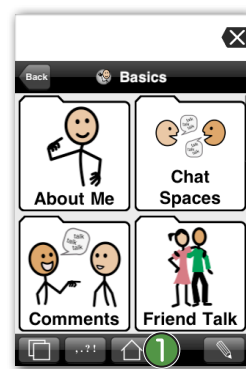


Tap on the Home button 1 if you are not already on the Home page.

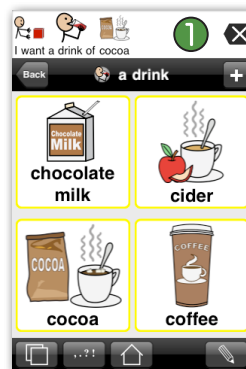


Tap on the *I want* category, tap on the *a drink* category, tap on the *a drink of* item and then tap on the *cocoa* item. You will notice that nothing is spoken when you tap the buttons.

Tap on the message window 1 to hear the message spoken.

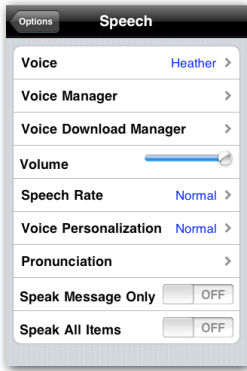


Go back to the Speech options. (If you don't remember how to get there, please see the instructions on the first page)

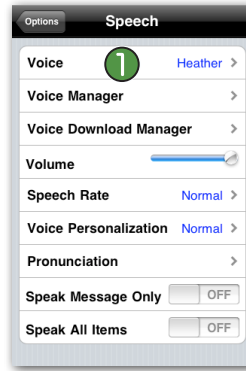


Go back to the Speech options. (If you don't remember how to get there, please see the instructions on the first page)


Tap on Voice. 1

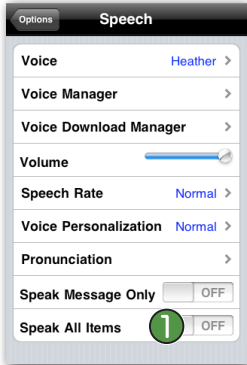


Tap on the Speak Message Only ON/OFF toggle so it is in the OFF position again. Tap on the Speak All Items ON/OFF toggle 1 so it is in the ON position.

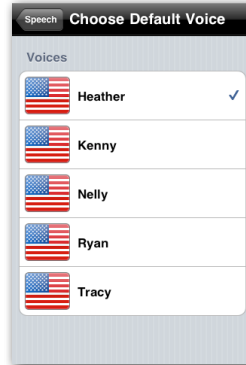

Here you can change the default voice. Tap another voice to hear the voice speak and select the one you prefer. Note that you can only switch languages once per 24 hour period.

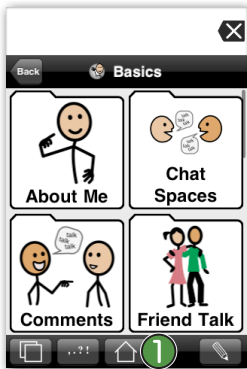

Tap on the Home button if you are not already on the Home page.

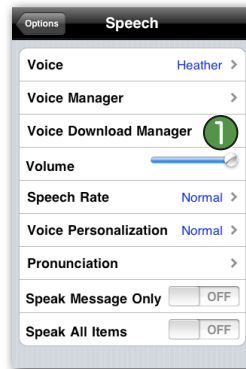


Tap Speech on top of the Choose Default Voice screen to go back to the Speech options.

Tap on Voice Download Manager. 1 Here you can download more voices. The download will continue in the background if you leave the download page.




Tap on the I want category, tap on the a drink category, tap on the a drink of item and then tap on the cocoa item. You will notice that now every button is spoken when you tap the buttons.

Tap on the message window 1 to hear the message spoken.

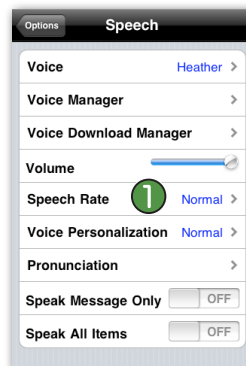


Tap Speech on top of the Voice Download Manager screen to go back to the Speech options.

Tap on Speech Rate. 1

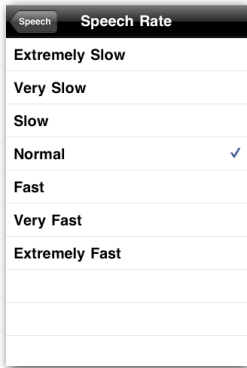


Here you can change the Speech Rate. Tap Extremely Slow. You will now hear the voice speak extremely slow. Try a few different rates and select the one you prefer.

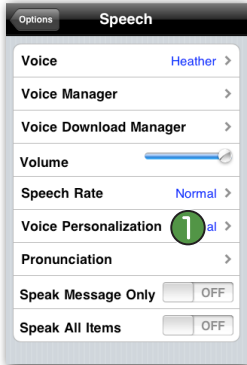


Tap Speech on top of the Speech Rate screen to go back to the Speech options.

Tap Voice Personalization. 1

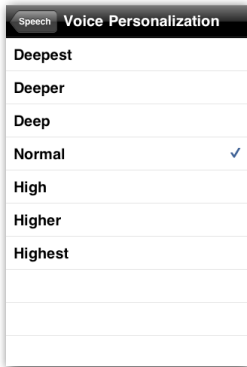


Here you can change the tone of the voice. Tap on Deepest. You can now hear the voice in a deeper tone. Try different tones and select the one you prefer.

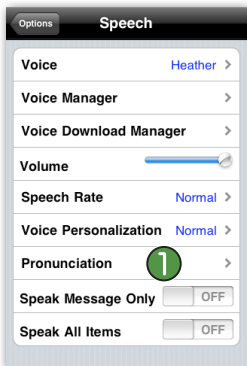

Tap Speech on top of the Voice Personalization screen to go back to the Speech options.

Tap on Pronunciation. 1

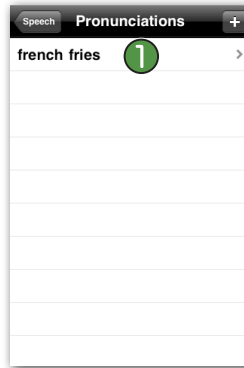

This screen shows you a list of the words and terms for which a pronunciation correction is in place. By default, for American English, you will see the term "french fries" listed. 1

Tap on french fries.



Here you can see the details of the pronunciation for this term.



If you want to change the pronunciation of a word, enter the new pronunciation under the Pronunciation section. 1

For more advanced pronunciation editing, you can use phonetic codes.



You can find specific instructions on how to change the pronunciation and on how to use phonetics in the tutorial Modifying Pronunciation: http://www.proloquo2go.com/files/Proloquo2Go_Tutorial-Modifying_pronunciation.pdf