




Using Recents to Create an Item


Proloquo2Go (iPad, iPhone and iPod touch). In this tutorial you will learn how to use Recents View to create new items.

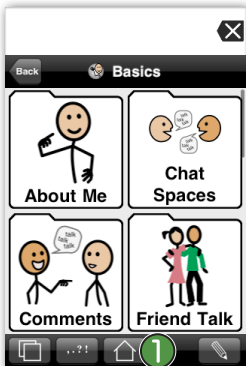


Navigate to the Proloquo2Go app on your device and start Proloquo2Go by tapping its icon. If you are not on the Home screen, tap on the Home button **1** on the toolbar.




Tap on the *I want* category **1** and then on the *to do something fun* item. Tap on the message window to hear the message. **2** Tap Clear to clear the screen. Tap the Home button to go back to the home screen.

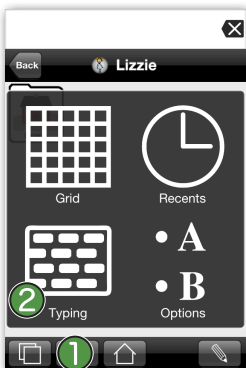

Tap on the Views button **1** on the toolbar and then tap on the Typing button. **2**

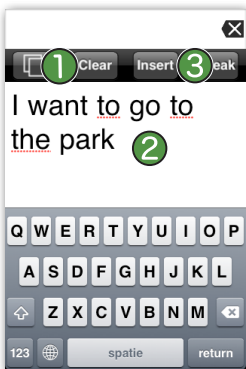


If the text screen still has some text, tap on the Clear button. **1** Type "I want to go to the park" in the text area. **2** Tap on the Insert button. **3**

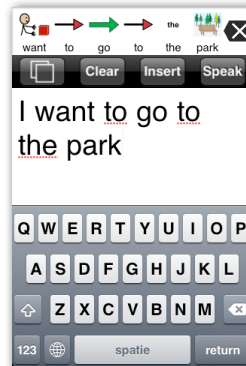

Notice how the text is now symbolized. Tap on the message window to hear the message and then on the Clear button.



Double-tap on the delete button to clear the message window. Type "I want to go to the Zoo" in the text area. Tap on the Insert button, the text will symbolize again. Tap on the message window to hear the message.

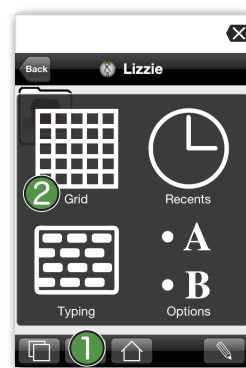


Tap on the Views button **1** from the toolbar and tap on the Grid view. **2**




Tap the Spaces category **1** and then tap on the User Spaces category.




Tap on the Edit button ① and then on the add button (+). In the Text to speak area, type in Activities. Tap on the Done button of the keyboard.

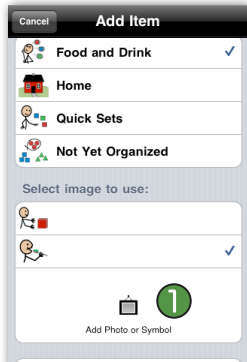


Tap on Add Photo or Symbol. ① Tap on Choose a Symbol. In the search window, type *activities*. Tap on the symbol “activities”.

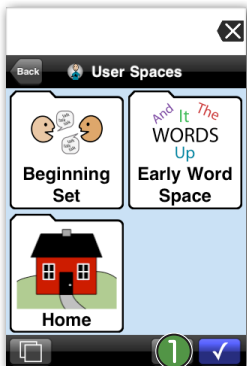

Scroll down and tap on Add as new Category. Tap on Create New Category. Tap on the blue exit-editing button ① to exit Edit Mode.

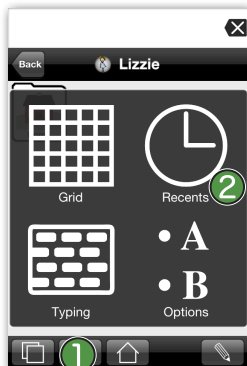


Tap on the Activities category and then tap on the Views button on the toolbar. Then tap on Recents View. ②

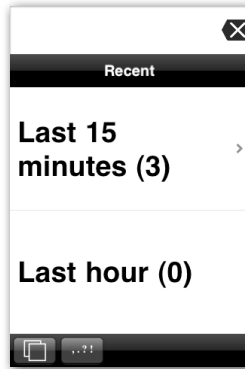


Tap on Last 15 Minutes.



Tap on the plus button ① beside the phrase “I want to do something fun”. Scroll down and tap on the Add button. Do the same thing for the phrases “I want to go to the park” and “I want to go to the Zoo”.

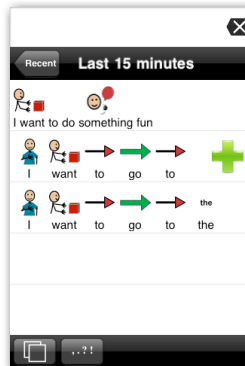

You will notice that every time you add an item using Recents, the plus symbol disappears because you have added the item to the vocabulary.

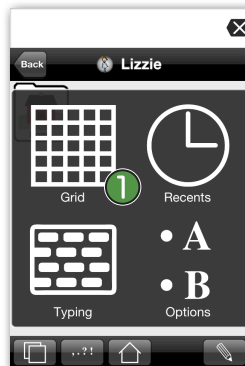

Tap the Views button on the toolbar and then tap Grid View. ① The items have now been added to your Activities category.