




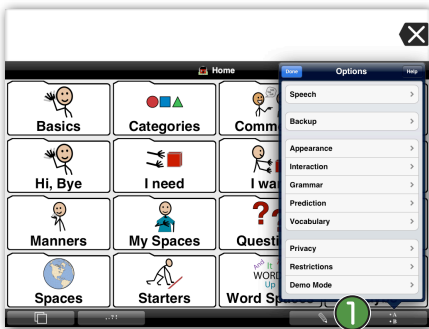
Working with Voices


Proloquo2Go (iPad, iPhone and iPod touch). In this tutorial you will learn all about the Voice options Proloquo2Go offers: Changing, Customizing and Downloading Voices.


Changing a Voice


 Launch Proloquo2Go by tapping on its icon. Go to the Options view.


On an iPod touch or iPhone, the Options View can be accessed by tapping the views toolbar button at the bottom left of the screen, and then tapping the Options button ① in the Views display.

 *On iPad, the Options button ① is the right-most button on the toolbar.*


 Tap on the Speech item. ①

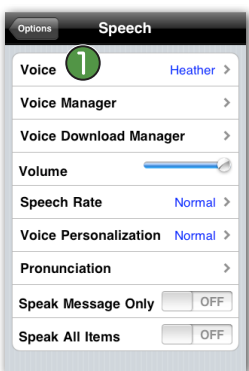
 Tap on the Voice item. ① Tap on any of the voices to select a different voice.

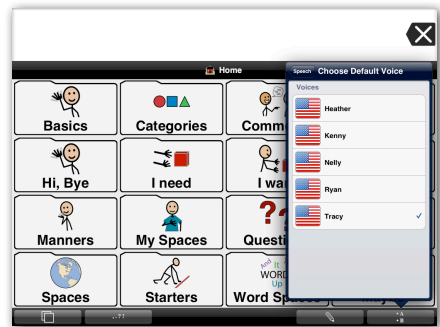 *On an iPad, tap anywhere outside the Speech screen to exit Options.*

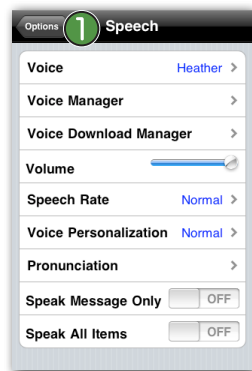 *On iPhone or iPod touch, tap on the Options button ① at the top of the screen to go back out to the Options window. There tap Done to exit the Options.*

Customizing Voices


 Tap on the Speech Rate section. ①

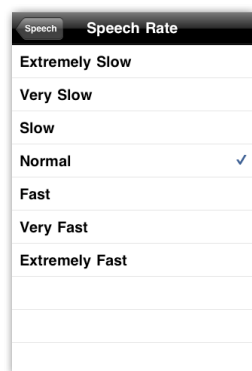 Here tap on the Slow rate. You will be able to hear a sample of how the voice will sound at a slow rate.


Then tap on the Very Fast rate. You will now hear a sample of how the voice will sound at a very fast rate. Select the rate you prefer.




Tap on the Options button at the top of the screen to go back out to the Options window again. Tap on the Voice Personalization section.

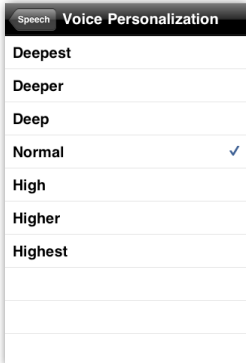

Tap on “Deeper” to hear the voice in a deeper tone. Tap on “Higher” to hear the voice in a higher tone. Select the tone you prefer.

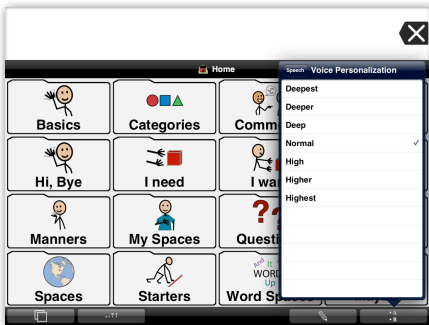

On an iPad, tap anywhere outside the Speech screen to exit Options.

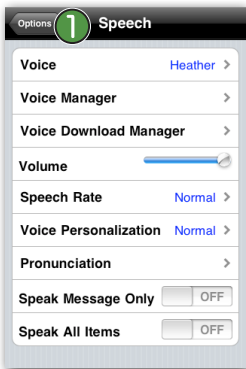


On iPhone or iPod touch, tap on the Options button 1 at the top of the screen to go back out to the Options window. There tap Done to exit the Options.


Downloading Additional Voices




From the Options screen, tap on the Download More Voices section.

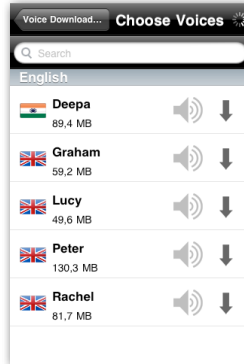

Tap Download More Voices and select the voice you would like to download.

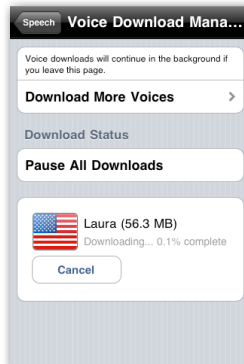

Note: To download additional voices, your device needs to be connected to a WiFi network. Downloading voices can take some time.

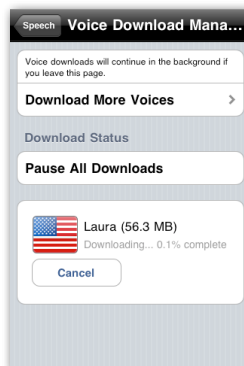

Sleep will interrupt downloads, so make sure you connect your device to a computer or power supply to prevent it from going into sleep mode. The download will continue in the background if you leave the download page.