

Adjusting Restrictions and Privacy on iPad



Restrictions

The first screenshot shows the main Options menu with various tabs like Sentence Prediction, Quick Talk, and Help. The second screenshot shows the Restrictions screen where 'Allow Editing' and 'Allow Sharing' are toggled ON. The third screenshot shows the Lock Options screen where 'Lock' is toggled OFF.

1 Accessing the Options

Tap the *Options* button on the top-right of the Text Pad to access the Options.

2 Going to Restrictions

In the Options, tap *Restrictions*. Note that these restrictions will only apply to the current user.

3 Allow Editing

In the Restrictions Options, toggle *Allow Editing* to OFF to prevent the user from accessing the Edit feature. You will notice that the Edit button (the pencil) on the top-right of the Text Pad will immediately be removed.

4 Allow Sharing

Toggle *Allow Sharing* to OFF. This will prevent the user from sharing text through Mail, Messages, Facebook, Twitter and/or AirDrop (iOS 7 only). You will notice that the Sharing button is immediately removed.

5 Lock Options

In the Options, tap *Lock Options*. Here you can restrict access to the entire Options. Note that these settings apply to all users.

6 Enabling Lock

In the Lock Options, toggle *Lock* to ON to enable password protection of the Options. Enter and save your password when prompted. The Options are now password protected.

If you have forgotten or lost your password, go to *Support* in the Options, tap *Email Support* and ask for unlock instructions. Make sure you are connected to Internet to send the email.


7 Change Password

If you want to change the password with which the Options are protected, tap *Change Password*. Here you will be able to enter and confirm your new password.

Adjusting Restrictions and Privacy on iPad



Privacy

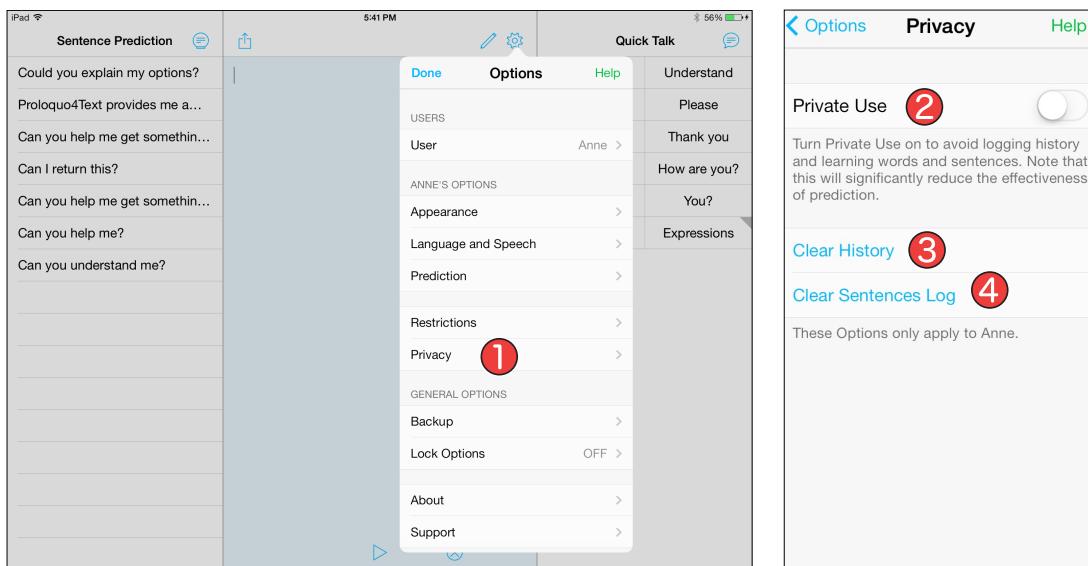

1 Going to Privacy

When using Proloquo4Text with multiple users or when you want to write something very private and do not want anyone to see what you said, go to *Privacy* in the Options.

2 Private Use

In the Privacy Options, toggle *Private Use* to ON to avoid logging history and learning words and sentences. Note that this will significantly reduce the effectiveness of prediction.

3 Clear History

Tap *Clear History* if you have forgotten to activate Private Use and want to assure the privacy of something you just typed.

4 Clear Sentences Log

Tap *Clear Sentence Log* if you have forgotten to activate Private Use and want to assure the privacy of something you typed. Note that this will reduce the effectiveness of sentence prediction significantly.