



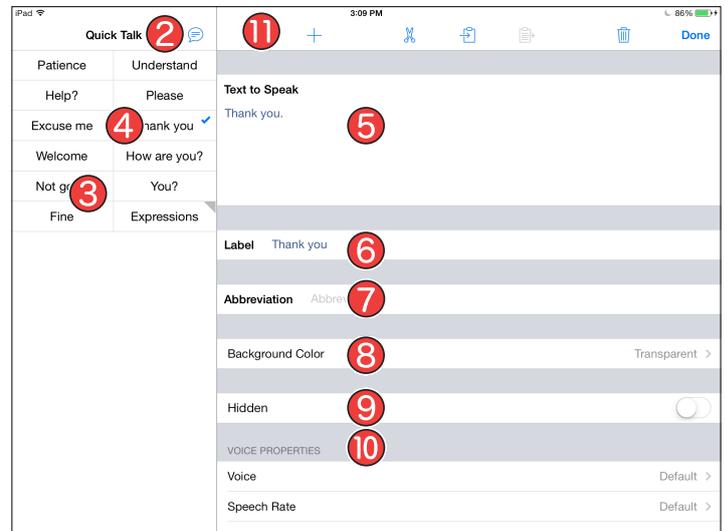

## Quick Talk

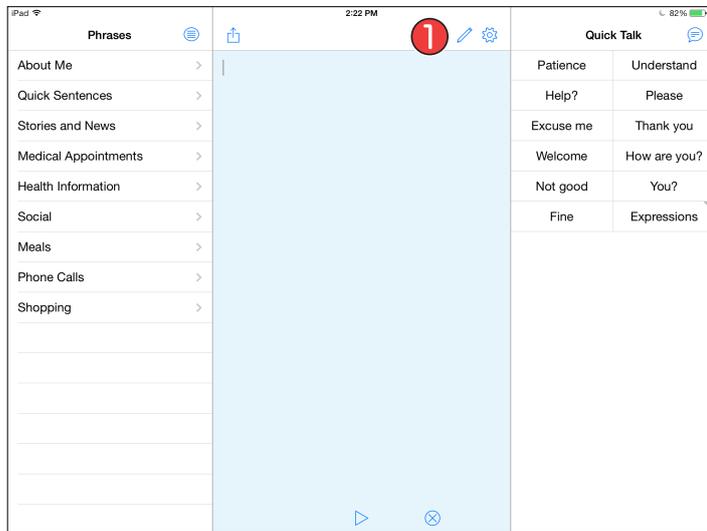

### 1 Accessing Edit mode

Tap the *Edit button* (the pencil) on the top-right of the Text Pad to access the Options.

### 2 Going to Quick Talk

In Edit mode, tap the Quick Block icon to swap between Quick Blocks until you arrive at Quick Talk. Here you can see the Quick Talk Categories and Items. You can recognize a Category by the little triangle in the top-right.

### 3 Repositioning Categories & Items

To reposition a Category or Item in Grid View, tap and hold on it until it pulses and then move it to a new position.

### 4 Editing Categories & Items

Tap the Category or Item you want to edit. A checkmark will confirm your selection. To open a Category in Grid View, double tap it.

### 5 Text to Speak

In the Editor, tap in the Text to Speak field to show the keyboard and edit the Text to Speak.

### 6 Label

Tap in the *Label* field to change the text of the label. The label is the title used for Categories and Items in the Quick Talk block. It can be the same as or different from the Text to Speak.

### 7 Abbreviations

Abbreviations can be used to speed up typing. Tap in the Abbreviation field to add an abbreviation for an Item. When you type an abbreviation, the text will be spoken/expanded.

### 8 Background Color

Tap *Background Color*. You can choose between all kinds of color scales. This feature can be used to make certain Categories and Items more visible.

### 9 Making Categories & Items hidden/visible

Toggle *Hidden* to ON or OFF to make Categories and Items hidden or visible. When Hidden is ON, a Category or Item will only be visible in Edit mode. Note that abbreviations still work even when an Item is hidden.

### 10 Voice Properties



In *Voice Properties*, you can change the Voice, Speech Rate (faster or slower), Voice Personalization (higher or deeper) and Relative Volume of a Category or Item.

### 11 Using the editing tools

Use the tools on the top toolbar to Add, Cut, Copy, Paste or Delete a Category or Item. Note that buttons can be copied from Proloquo2Go. For more info, see: [bit.ly/CopyFromP2GtoP4T](http://bit.ly/CopyFromP2GtoP4T)



## Phrases

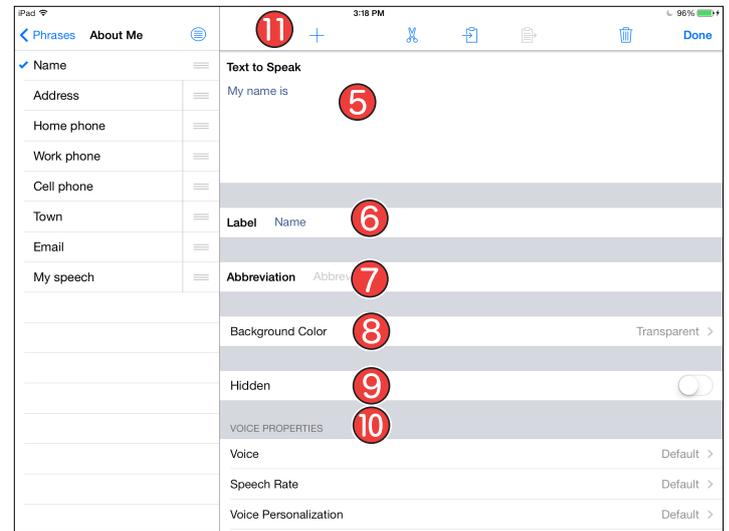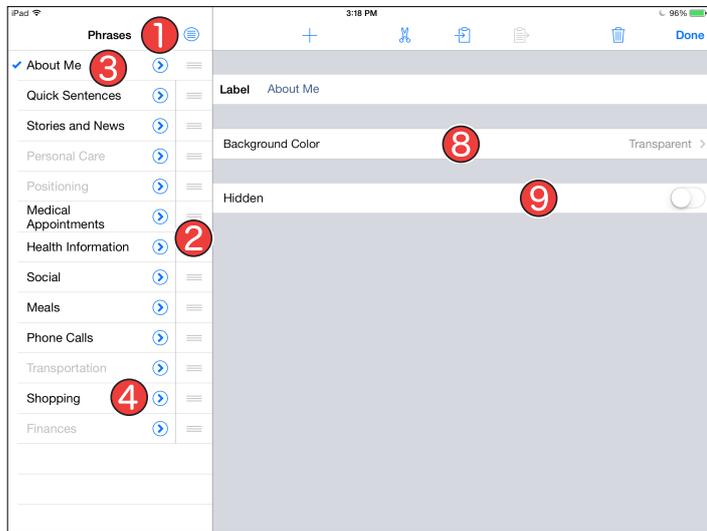

### 1 Going to Phrases

In Edit mode, tap the Quick Block icon to swap between the Quick Blocks until you arrive at Phrases. Here you can see all the Categories. Some are greyed out. This means they are not visible for the user outside of Edit mode.

### 2 Repositioning Categories & Items

To reposition Categories and Items, tap and hold on the move bars and drag the Category or Item up or down.

### 3 Editing Categories

Tap a Category to edit it. A checkmark will confirm your selection.

### 4 Editing Items

Tap the blue arrow to open a Category and select an Item you want to edit. A checkmark will confirm your selection.

### 5 Text to Speak

Tap anywhere in the Text to Speak field to show the keyboard and edit the Text to Speak..

### 6 Label

Tap in the *Label* field to change the text of the label. The label is the title used for Categories and Items in the Phrases block. It can be the same as or different from the Text to Speak.

### 7 Abbreviations

Abbreviations can be used to speed up typing. Tap in the Abbreviation field to add an abbreviation for an Item. When you type an abbreviation, the text will be spoken/expanded.

### 8 Background Color

Tap *Background Color*. You can choose between all kinds of color scales. This feature can be used to make certain Categories and Items more visible.

### 9 Making Categories & Items hidden/visible

Toggle *Hidden* to ON or OFF to make Categories and Items hidden or visible. When Hidden is ON, a Category or Item will only be visible in Edit mode. Note that abbreviations still work even when an Item is hidden.

### 10 Voice Properties

In *Voice Properties*, you can change the Voice, Speech Rate (faster or slower), Voice Personalization (higher or deeper) and Relative Volume of a Category or Item.

### 11 Using the editing tools

Use the tools on the top toolbar to Add, Cut, Copy, Paste or Delete a Category or Item. Note that buttons can be copied from Proloquo2Go. For more info, see: [bit.ly/CopyFromP2GtoP4T](http://bit.ly/CopyFromP2GtoP4T)